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7.3 Freilegen des Implantats und Weichgewebemanagement
Die CAMLOG® Guide System ist part of a total concept and must only be used with the SCREW-LINE Implants CAMLOG® Guide belonging to this system and the related CAMLOG® instruments and components in accordance with the directions and recommendations of the manufacturer. All components of the CAMLOG® Guide System are matched precisely to one another. The use of components and instruments foreign to the system can affect the function and safety of the CAMLOG® Guide System. CAMLOG does not warrant nor provide replacement services when non-system components are used. Therefore, you should use surgical, prosthetic components and instruments from CAMLOG exclusively. All components of the CAMLOG® implant System are optimally matched to one another and each forms part of a complete system.

Die CAMLOG® Guide System is part of a total concept and must only be used with the SCREW-LINE Implants CAMLOG® Guide belonging to this system and the related CAMLOG® instruments and components in accordance with the directions and recommendations of the manufacturer. All components of the CAMLOG® Guide System are matched precisely to one another. The use of components and instruments foreign to the system can affect the function and safety of the CAMLOG® Guide System. CAMLOG does not warrant nor provide replacement services when non-system components are used. Therefore, you should use surgical, prosthetic components and instruments from CAMLOG exclusively. All components of the CAMLOG® implant System are optimally matched to one another and each forms part of a complete system.

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NOTE: ALTEATEC GmbH/CAMLOG Biotechnologies AG waive all liability for the performance of planning and its transfer to the drilling template. **Warning! Before using the CAMLOG® Guide System, the user must be familiar with the 3D planning system as well as the milling system if one is used.**

NOTE: ALTEATEC GmbH/CAMLOG Biotechnologies AG waive all liability for the performance of planning and its transfer to the drilling template. **Warning! Before using the CAMLOG® Guide System, the user must be familiar with the 3D planning system as well as the milling system if one is used.**

NOTE: ALTEATEC GmbH/CAMLOG Biotechnologies AG waive all liability for the performance of planning and its transfer to the drilling template. **Warning! Before using the CAMLOG® Guide System, the user must be familiar with the 3D planning system as well as the milling system if one is used.**

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[English]

CAUTION! Before using the product, please read the following instructions.

1 Product Description
The CAMLOG® Guide System includes:

- Laboratory instruments for fabricating a drilling template and a preoperative interim prosthesis on a master cast.
- Surgical instruments for template-guided endosseous or periodontal supported implant bed preparation and implant insertion.
- SCREW-LINE Implants CAMLOG® Guide.

When using suitable 3D planning software with an appropriate positioning guide, including a milling spindle, fabricating a drilling template is made possible with the aid of the CAMLOG® Guide System laboratory instruments. As an alternative to the clinician or laboratory technician fabricating a drilling template on a positioning guide, some manufacturers of planning systems offer centralized production of the drilling template. In this case, the laboratory instruments for fabricating the template are not required.

Once fabricated, this drilling template is used to:
a) Position lab analogs during preoperative cast fabrication
b) Guide surgical instruments of the CAMLOG® Guide System during implant bed preparation
c) Guide SCREW-LINE implants CAMLOG® Guide during their insertion.

SCREW-LINE Implants CAMLOG® Guide, are endosseous implants, available in different lengths and diameters. They are implanted surgically in the maxillary or mandibular bone for the purpose of anchoring functional esthetic oral restorations in partially or fully edentulous patients. The prosthetic treatment is then completed with single crowns, bridges or full dentures, which are fastened to the SCREW-LINE implants CAMLOG® Guide with adequate attachments.

NOTE: SCREW-LINE Implants CAMLOG® Guide differ from conventional SCREW-LINE implants in a modified insertion post. To insert the SCREW-LINE implants CAMLOG® Guide, the associated driver for CAMLOG® Guide implants, short, manual/wrench, Ø 3.8/4.3 mm, Art. No. JB300.0012 is required.

The healing options of the SCREW-LINE implants CAMLOG® Guide include:

- Submerged healing with a cover screw
- Transgingival healing with a healing cap, ball abutment, ball abutments, ball abutments, ball abutments, or with double crowns.
- Transgingival healing with a long-term temporary restoration as immediate restoration.

2 Indications
Template-guided implant bed preparation and insertion of SCREW-LINE Implants CAMLOG® Guide after prior planning with a suitable 3D planning system. SCREW-LINE Implants CAMLOG® Guide are used in indications for oral endosseous implants in the maxilla and mandible for the functional and/or esthetic rehabilitation of partially or fully edentulous patients.

3 Contraindications
Insufficient bone volume, insufficient soft tissue coverage and/or inadequate bone quality, local root remnants, bone and wound healing disorders, local infection at the implantation site, severe refractory functional disorders, uncontrolled diabetes mellitus, long-term immunosuppressant drug therapy, connective tissue/collagen diseases, hematological diseases (e.g. leukemia, hemophilia), intra-oral infections or malignancies, uncontrolled para-functional habits, unreatable occlusal or articulation disorders, severe psychological disorders, xerostomy and sensitivity to titanium.

Precautions
It is important to obtain a thorough medical history from the patient and if necessary from the general practitioner prior to conducting implant surgery in order to determine if conditions exist that will 1) make implant placement difficult because of anatomical conditions, 2) create a significant surgical problem or general risk, 3) impair healing capacity and/or osseointegration, or 4) lessen the likelihood of proper hygiene and/or maintenance of the implant, abutments and fixtures. The following are examples of conditions to be considered in each category. Some of these conditions may be relevant to more than one category. If any of these conditions or a combination of conditions are severe or uncontrolled, dental implants should not be used.

Anatomical Conditions
Incomplete jaw development, difficult anatomical bone relationships, previously irradiated bone, temporomandibular joint disease and treatable jaw conditions.

Surgical and General Risks
Severe systemic diseases, reduced immune response and leukocytic disorders that increase the risk of infection, and/or healing delays, osteoporosis, anticoagulation therapy/anticoagulant diabetes, arteriosclerosis and CVA, hypertension, cardiac infarct, diseases requiring periodic uses of steroids, hepatitis, diabetes mellitus and pregnancy.

Impaired Healing Capacity
Bone metabolism disorders, any disease that affects bone regeneration or microcirculation of the blood, rheumatic diseases and abuse of drugs including alcohol and tobacco.

Maintenance
Inadequate patient compliance, inadequate oral hygiene, periodontitis, bruxism, loss of proper functionality and oral mucosal changes.

Implants with a diameter of 3.8 mm are not to be used with angled abutments in the molar region.

4 Side Effects
In rare cases, it is possible that osseointegration does not occur. In such an event, consult your local CAMLOG representative.

Warning! Activities requiring strenuous bodily effort must be avoided immediately after the insertion of dental implants.

Possible complications after dental implant insertion may include:

Temporary symptoms
Pain, swelling, speech difficulties, gingival inflammation.

Prolonged symptoms
Chronic pain associated with the dental implant, permanent paresthesia, dysesthesia, nerve damage, exfoliation, hyperplasia, localized or systemic infection, oronasal or oronasal fistulas, loss of maxillary/mandibular ridge bone, negative impact on adjacent teeth, irreversible damage to adjacent teeth, fractures of the implant, jaw bone or restoration, esthetic problems.

5 General Safety Instructions and Warnings
An improper procedure in surgery and/or prosthetics can lead to implant damage and/or bone loss. The CAMLOG® Guide System should only be utilized by dentists, physicians, surgeons and dentists who, at the point in time that the product is placed on the market. This does not exempt the user from the responsibility to personally test the product for suitability for the intended purpose, indication and procedure. Handling and use of the product take place outside of our control and is the direct responsibility of the user. All liability for damages resulting from such use is waived.

Every patient must be thoroughly examined and evaluated on topographic, psychological, and physical parameters, including the condition of the teeth and edentulous areas, and the condition, which might affect the final outcome. Close collaboration between surgeon, prosthodontist and dental technician is essential for success.

The CAMLOG® Implant System and the related procedures have been developed and clinically tested by experts in the field. Detailed information on the choice of suitable implants and abutments, on treatment plans and the application of dental implants is contained in the use informations of CAMLOG Biotechnologies AG.

The CAMLOG® Guide System is part of a total concept and must only be used with the SCREW-LINE Implants CAMLOG® Guide belonging to this system and the related CAMLOG® instruments and components in accordance with the directions and recommendations of the manufacturer. All components of the CAMLOG® Guide System are matched precisely to one another. The use of components and instruments foreign to the system can affect the function and safety of the CAMLOG® Guide System. CAMLOG does not warrant nor provide replacement services when non-system components are used. Therefore, you should use surgical, prosthetic components and instruments from CAMLOG exclusively. All components of the CAMLOG® implant System are optimally matched to one another and each forms part of a complete system.

An improper procedure in surgery and/or prosthetics can lead to implant damage and/or bone loss. The CAMLOG® Guide System should only be utilized by dentists, physicians, surgeons and dentists who, at the point in time that the product is placed on the market. This does not exempt the user from the responsibility to personally test the product for suitability for the intended purpose, indication and procedure. Handling and use of the product take place outside of our control and is the direct responsibility of the user. All liability for damages resulting from such use is waived.

Every patient must be thoroughly examined and evaluated on topographic, psychological, and physical parameters, including the condition of the teeth and edentulous areas, and the condition, which might affect the final outcome. Close collaboration between surgeon, prosthodontist and dental technician is essential for success.

The CAMLOG® Implant System and the related procedures have been developed and clinically tested by experts in the field. Detailed information on the choice of suitable implants and abutments, on treatment plans and the application of dental implants is contained in the use informations of CAMLOG Biotechnologies AG.

The CAMLOG® Guide System is part of a total concept and must only be used with the SCREW-LINE Implants CAMLOG® Guide belonging to this system and the related CAMLOG® instruments and components in accordance with the directions and recommendations of the manufacturer. All components of the CAMLOG® Guide System are matched precisely to one another. The use of components and instruments foreign to the system can affect the function and safety of the CAMLOG® Guide System. CAMLOG does not warrant nor provide replacement services when non-system components are used. Therefore, you should use surgical, prosthetic components and instruments from CAMLOG exclusively. All components of the CAMLOG® implant System are optimally matched to one another and each forms part of a complete system.

Use of the surgical instruments of the CAMLOG® Guide System must take into account the selected implant diameter and implant length. Surgical instruments of the CAMLOG® Guide System must be used exclusively with the guiding sleeves CAMLOG® Guide when preparing the implant bed for SCREW-LINE Implants CAMLOG® Guide. Their use with any other implant line or for an unintended diameter or length can lead to the mechanical failure of system components, tissue injury, or esthetically unsatisfactory results.

Warning! Sterile CAMLOG® Guide products must not be used if:

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